Prior to use, carefully read the instructions for use.  EN

## FIT CHECKER™ ADVANCED

VPES SILICONO PER CHECKING FIT

For use only by a dental professional in the recommended indications.

#### RECOMMENDED INDICATIONS

To check the fit of dental prosthetic appliances.

- Crown: margin, proximal surface, occlusal surface
- Precast post: fit of post
- Inlay: margin, proximal surface
- Denture: mucosal surface

#### CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to silicone.

#### PRECAUTION

When extruding or mixing FIT CHECKER ADVANCED, care should be taken to avoid mixing or contact with the following materials which may delay or prevent setting: Catalyst for condensation silicone impression materials, polysulfid impression materials, eugenol materials, sulfur, latex, oil, acrylates, glycerol and spray-type or ointment-type topical surface anesthesia (e.g. lidocaine).

#### DIRECTIONS FOR USE MIXING

Squeeze out equal lengths of base and catalyst onto the mixing pad (1 cm for crown or inlay, 2-3 cm for partial denture, 3-4 cm for complete lower denture, 4-5 cm for complete upper denture). Mix thoroughly for 20 seconds.
Note:

- To extend working time, add retarder before mixing as shown in the table below. Retarder provided with FIT CHECKER or FIT CHECKER II CANNOT be used with ADVANCED.
- Continue squeezing tube until replacing the cap to avoid the air being sucked into the tube. Air in the tubes may cause solid-liquid separation.
- After use, close tubes with their original caps.

Number of drop of retarder to 5cm of base and catalyst	Working time (min)	Minimum time in mouth (min)
0	1 min.	
3	1 min. 10 sec.	1 min.
5	1 min. 20 sec.	

At 23°C, humidity 50%

#### CARTRIDGE LOADING AND DISPENSING

- Lift the release lever of CARTRIDGE DISPENSER II and pull the piston plunger all the way back into the dispenser. Check the hot end of the dispenser and load the cartridge, ensuring that the V-shaped notch on the flange of the cartridge is facing down. Push the cartridge holder down to hold the cartridge firmly in place.
- Lift the release lever and push the piston plunger forward until it engages into the cartridge.
- Remove the cartridge cap by rotating 1/4 turn counter-clockwise. Tilt the cap downward and peel it away from the cartridge. Gently squeeze the cartridge handle to extrude a small amount of material from the two openings at the end of the cartridge. Make sure that base and catalyst come out evenly.
- Align the V-shaped notch on the rim of the MIXING TIP II to the V-shaped notch between the cartridge barrels. Push firmly to attach the mixing tip. Then rotate the colored collar of the mixing tip 1/4 turn clockwise to the end of the cartridge. The dispenser is now ready for use.
- Squeeze the handle several times to extrude the material. After use, do not remove the MIXING TIP II as this will become the storage cap until next use. When replacing the MIXING TIP II, rotate the collar on the mixing tip 1/4 turn counter-clockwise to align the V-shaped notch on the cartridge. Tilt the mixing tip downward and peel it away from the cartridge.
- Remove the old mixing tip prior to next use. Before attaching a new tip, gently extrude a small amount of material to ensure that base and catalyst are flowing evenly from both openings. If the materials should fail to extrude, remove any hardened materials from the end of the cartridge.
- To replace the cartridge, lift the release lever and retract the piston plunger fully. Remove the empty cartridge by lifting the cartridge holder and load a new one into the dispenser.

#### FIT CHECKING

- Apply the mixture to the internal surface of the prosthetic appliance and seat in the mouth. Working time is 1 minute from start of mix at 23°C (73°F). Hold in position for 1 minute until the material is set.
- Remove the prosthesis using a suitable instrument. Check the thickness of FIT CHECKER ADVANCED to assess the fit of the prosthesis. A suitable fit is shown by being uniformly covered with a thin film. If there is a spot where the film appears to be extremely thin or thick, it should be corrected using standard techniques.
- After assessment, remove the material and clean the prosthesis.
Note:
  - Do not use FIT CHECKER ADVANCED on the surface relined with silicone based soft relining material as it would be difficult to remove.
  - FIT CHECKER ADVANCED is highly flowable material. Care should be taken to ensure the material does not flow into the patient's throat.
  - Avoid desiccation of tooth and surrounding tissue when applying FIT CHECKER ADVANCED.
  - FIT CHECKER ADVANCED has unique chemistry that allows it to polymerize faster in the oral cavity than at room temperature. Therefore to determine setting, examine material in the mouth since material on bench will yield a false result.
  - Higher temperatures shorten the working time and lower temperatures increase working time.

#### COLOR

White

#### STORAGE

Recommended for optimal performance, store at room temperature away from direct sunlight (15-25°C, 59-77°F).

#### PACKAGES

- Tube
  - 1-1 Pkg: 55 g (42 mL) base, 50 g (42 mL) catalyst, 3 g retarder and mixing pads
- Cartridge
  - Two cartridge package: Cartridge 62 g (48 mL) x 2, MIXING TIP II SS x 3, MIXING TIP II SS x 3

#### CAUTION

- In case of contact with oral tissue or skin, remove and flush with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Do not mix FIT CHECKER ADVANCED with components of any other products.
- Avoid getting material on clothing as it is difficult to remove.
- Take care to avoid ingestion of the material.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.
- In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

- http://www.gceurope.com or for The Americas
- http://www.gcamerica.com

They can also be obtained from your supplier.

#### CLEANING AND DISINFECTING

MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Do not use if damaged.
**DO NOT IMMERS**E. Thoroughly clean device to prevent drying and accumulation of contaminants.
Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:

- https://ec.europa.eu/growth/sectors/medical-devices/contacts\_en as well as to our internal vigilance system: vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

Last revised: 01/2020

**IGC** **Rx Only**

MANUFACTURED BY
**GC CORPORATION**
76-1 Hasunuma-cho, Habashi-Ku, Tokyo 174-8685, Japan
RESponsible Manufacturer in CANADA:
**GC AMERICA INC.**
3737 West 127th Street, Aliso, IL 60803 U.S.A.

DISTRIBUTED BY
**GC CORPORATION**
76-1 Hasunuma-cho, Habashi-Ku, Tokyo 174-8685, Japan
**EU** : GC EUROPE N.V.
Houtlandplant Hastenrode 120a, Houtlandveerbaan 33, B-3010 Leuven, Belgium TEL: +32 16 74 10 00
**GC AMERICA INC.**
2727 The Americas Drive, Aliso Viejo, CA 92656 U.S.A. TEL: +1 760 597 0900 www.gcamerica.com

**GC SOUTH AMERICA**
Rua Heliadora, 309, Santana - São Paulo, SP, BRASIL. CEP: 02022-051 - TEL: +55-11-2925-0965
CNPJ: 08.278.998/0001-61
**GC AUSTRALASIA DENTAL PTY. LTD.**
1753 Botany Rd, Bankmeadows, NSW 2019, Australia TEL: +61 2 9301 8200

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