

G-Premio BOND Technique Guide



G-Premio BOND

Select etching mode



Self-etch



10-15 SEC.

Selective etch

Etch enamel only for 10-15 sec., rinse for 5 sec. and dry.



10-15 SEC.

Total-etch

Etch for 10-15 sec., rinse for 5 sec. and dry.

Bonding procedure




10 SEC.

1. Apply G-Premio BOND and wait 10 sec.



5 SEC.

2. Dry for 5 sec. (MAX air pressure).

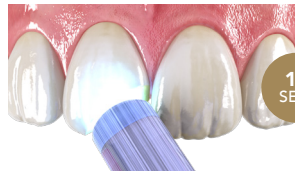

10 SEC.

3. Light-cure for 10 sec.



4. Start placing composite.

G-Premio BOND Technique Guide



Intraoral repair



1. Roughen surface.
Rinse and dry.



2. For all glass ceramics and hybrid ceramics, apply G-Multi PRIMER. Dry.



3. Apply G-Premio BOND to all surfaces and wait 10 sec.



4. Dry for 5 sec.
(MAX air pressure).



5. Light-cure for 10 sec. Begin composite application to complete repair.

Hypersensitivity treatment



1. Apply G-Premio BOND. Wait 10 sec.



2. Dry for 5 sec. (MAX air pressure).



3. Light-cure for 10 sec.



4. Remove unpolymerised layer using a cotton pellet.

THESE PRODUCTS ARE NOT AVAILABLE FOR PURCHASE BY THE GENERAL PUBLIC. ALWAYS READ THE LABEL AND FOLLOW THE DIRECTIONS FOR USE.