EN
To use or use, carefully read the instructions for use.

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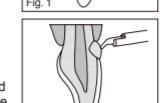
GRADIA™ DIRECT

LIGHT-CURED COMPOSITE RESTORATIVE
For use only by a dental professional in the recommended indications.

- RECOMMENDED INDICATIONS
A. GRADIA DIRECT ANTERIOR
1. Direct restorative for Class II, IV, V cavities.
2. Direct restorative for wedge-shaped defects and root surface cavities.
3. Restorative for veneers and diastema closure.
B. GRADIA DIRECT POSTERIOR (Röntgensicher)
1. Direkte Restauration von Kavitäten der Klassen II und V.

- CONTRAINDICATIONS
1. Pulp capping.
2. Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.
DIRECTIONS FOR USE
1. Shade Selection
Clean the tooth with pumice and water. Shade selection should be made prior to isolation. Select the appropriate GRADIA DIRECT shades by referring to the GRADIA DIRECT shade guide.

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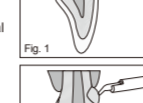
GC logo and contact information for various regions including Europe, North America, and Australia.

GC logo and contact information for various regions including Europe, North America, and Australia.

GC logo and contact information for various regions including Europe, North America, and Australia.

GC logo and contact information for various regions including Europe, North America, and Australia.

GC logo and contact information for various regions including Europe, North America, and Australia.

GC logo and contact information for various regions including Europe, North America, and Australia.