
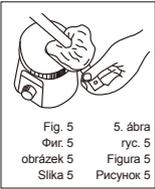
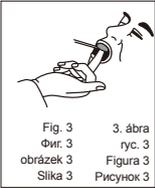
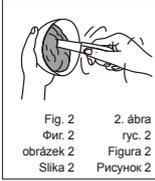
Prior to use, carefully read the instructions for use.

**EN**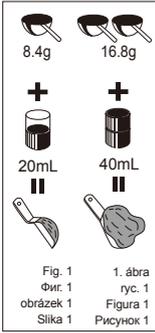

## AROMA FINE PLUS

ALGINATE IMPRESSION MATERIAL

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

Preliminary impressions.

Impressions for the fabrication of:

- models for temporary inlays, onlays, crowns and bridges
- orthodontic models
- study models

### CONTRAINDICATIONS

In rare cases, the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### DIRECTIONS FOR USE

	Normal set	Fast set
Mixing time	20-30 sec.	20-30 sec.
Total working time	90 sec.	70 sec.
Setting time	180 sec.	150 sec.
Minimum time in mouth	120 sec.	90 sec.

- Loosen the powder well before every use. Take the necessary amount of powder and water. Standard powder to water ratio is 1 spoon (8.4g) of powder and 1/2 cup (20mL) of water (Fig. 1).
- Mix for 20-30 seconds. Load the tray with the mixed material (Fig. 2).
- Seat the loaded tray in the mouth. Refer to above table for recommended total working times (Fig. 3).
- Wait for the material to set. Refer to above table for the minimum time in mouth.
- Remove impression and rinse under running tap water (Fig. 4).

- Note:
- The setting time includes the mixing time and the minimum manipulation time. This is based on the mixing at 23°C with distilled or deionized water.
  - Higher temperatures of the mixing water will shorten the setting- and total working time. Variations in hardness of the water may influence the setting time.

### DISINFECTION

The recommended disinfection method is to soak in a 0.5% sodium hypochlorite solution for 15 minutes. Refer to the respective manufacturer's instructions for use.

### MODEL FABRICATION

- Remove excess water before pouring cast. For the most accurate result, pour cast immediately after rinsing (Fig. 5).
- If the impression cannot be poured immediately, wrap the impression in a wet paper towel or store the wet impression in a tightly sealed plastic bag. **DO NOT STORE IN WATER.**
- AROMA FINE PLUS is compatible with type III and type IV stone (GC Fuji ROCK™ EP).

### STORAGE

Store in a cool dry place, in a tightly closed container (4-25°C) (39.2-77.0°F).

(Shelf life: 3 years from date of manufacture)

### CAUTIONS

- In case of contact of material or mixture with eyes, immediately flush with water and seek medical attention.
- When using material, wear a dust mask to avoid inhaling powder dust.
- After opening, transfer the contents into a container for exclusive use of the material.
- Do not mix with any other product.

Last revised : 05/2011

**CE**  
**'GC.'**

MANUFACTURED by  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

DISTRIBUTED by  
**GC CORPORATION**  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

**EU: GC EUROPE N.V.**  
Researchpark Haasrode•Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00

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