	30001153-NV4702 IFU StickRefix L WEP
	instrument. However, the colour change will not have adverse effects on the use of the instrument.

StickREFIX

SILICONE INSTRUMENTS

For use only by a dental professional in the indications for use.

INDICATIONS FOR USE

StickREFIX L and StickREFIX D silicone instruments are especially designed to facilitate the use of Stick and everStick fibre products. The use of the instruments in different indications is described in the instructions for use of Stick and everStick fibre products.

CLEANING OF StickREFIX SILICONE INSTRUMENTS

- RINSING**
Rinse the instrument with warm running water after use.
- CLEANING**
 - Cleaning by hand
 - Remove visible impurities with warm water and a soft brush
 - Use neutral detergent (pH ca. 7)
 - Rinse with warm running water
- STERILIZATION**

Only the StickREFIX D (long shape instrument) is to be used intra-orally and can undergo a sterilization cleaning process.

The round shape instrument does not need autoclave disinfection because it is not used in direct clinical application.

Autoclaving

- Autoclave, clean and properly packed in semi permeable autoclave pouch, StickREFIX D (long shape instrument) by using rubber program (recommended temperature 121 °C for 20 min)
- StickREFIX D can undergo a maximum of 15 sterilization cycle

NOTE: if disinfectants are used to clean silicone instruments, the suitability to silicone materials needs to be confirmed from manufacturer of the disinfectant. Remember to rinse properly the instrument after the disinfection.

PACKAGES

StickREFIX L, 3 + 3 pieces

everStickINTRO
8 cm everStickC&B; 8 cm everStickPERIO; 1x StickREFIX D silicone instrument; 2 ml G-ænial Universal Flo sprøyte A2;

everStickCOMBI
8cm everStickC&B; 8cm everStickPERIO; 30 cm² everStickNET; 5x everStickPOST Ø 1.2; 1x StickSTEPPER hand instrument; 1x StickREFIX D silicone instrument

everStickSTARTER KIT
8 cm everStickC&B; 8 cm everStickPERIO; 30 cm² everStickNET; 5x everStickPOST Ø 1.2; 6 mL GC Modeling Liquid bottle; 2 ml G-ænial Universal Flo syringe A2; 20 dispensing tips, 1 light protective cover; 1x StickREFIX D silicone instrument; 1x StickSTEPPER; 1x StickCARRIER

Important:

- Do not harm instruments by using hard brushes or engraved marks for identification, as these may crack the instrument surface
- Autoclaving may affect the colour of the

DISTRIBUTED by
 EU: GC EUROPE N.V., Researchpark Hasselt-Leuven 1240, Interleuvenlaan 33, B-3001 Leuven, Belgium TEL: +32 16 74 10 00
GC CORPORATION 76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
GC AMERICA INC. 3737 West 127th Street, Alsip, IL 60803 U.S.A. TEL: +1-708-597-0900 www.gc.dental/america
GC ASIA DENTAL PTE. LTD. No. 5 Tampines Central 1 #06-01 Tampines Plaza Singapore 529541 TEL: +65 6546 7588
GC AUSTRALASIA DENTAL PTY. LTD. 1753 Botany Rd, Banksmeadow, NSW 2019, Australia TEL: +61 2 9301 8200
GC SOUTH AMERICA Rua Heliodora, 399, Santana - São Paulo, SP, Brasil CEP: 02022-051, TEL: +55-11-2925-0965 CNPJ: 08.279.999/0001-61 RESP. TEC: Enick de Lima CRO/SP 100.866
RESPONSIBLE MANUFACTURER IN CANADA GC AMERICA INC. 3737 W. 127th Street, Alsip, IL 60803 U.S.A.
PRINTED IN BELGIUM

20 dispensing tips, 1 light protective cover; 1x StickREFIX D silicone instrument; 1x StickSTEPPER; 1x StickCARRIER

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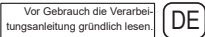
Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: https://www.gc.dental/europe or for The Americas https://www.gc.dental/america They can also be obtained from your supplier.

Undesired effects - Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medicaldevices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell Buckinghamshire MK16 8JS
United Kingdom

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