everX Posterior™

FIBER REINFORCED COMPOSITE FOR DENTIN REPLACEMENT
For use only by a dental professional in the indications for use.

- INDICATIONS FOR USE
everX Posterior is suitable for use as the reinforcing material for direct composite restorations, especially in large posterior cavities, for example:
1. Cavities including 3 surfaces or more
2. Cavities with missing cusps
3. Deep cavities (including class I, II and endodontically treated teeth)
4. Cavities after amalgam replacement
5. Cavities where onlays & inlays would also be indicated

- CONTRAINDICATIONS
1. Direct pulp capping
2. Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.
3. At least one horizontal dimension of the cavity should exceed 3 mm.
4. The product is not suitable for patients with a history of hypersensitivity to methacrylate monomers.
5. Do not use as final approximal and surface composite layer.

PRODUCT DESCRIPTION
everX Posterior is a light-cure, radio-opaque restorative material to be used intra-orally and classified as a Type 2 and Class 2 (Group 1) per ISO standard 4049. This material has a radiopacity equivalent to 2.5-3.0 mm of aluminum (dentin = 1 mm, enamel = 2 mm).

DIRECTIONS FOR USE
1. Shade Selection
Clean the tooth with pumice and water. Shade selection of the final composite enamel layer should be made prior to isolation.

- Cavity Preparation (Fig. 1)
Prepare cavity using standard techniques. Dry by gently blowing with oil free air.
2. Cavity Protection
For bonding everX Posterior to enamel and/or dentin, use a light-cured bonding system such as G-Premo BOND, G-anal Bond or G-BOND.
3. Bonding (Fig. 2)
For bonding everX Posterior to enamel and/or dentin, use a light-cured bonding system such as G-Premo BOND, G-anal Bond or G-BOND.
4. Placement of everX Posterior (Fig. 3) - see note for Class II and large cavities
Insert everX Posterior into the Unitip APPLIER 2 or equivalent.

- Curing (Fig. 4)
Optional, apply first a thin layer of flowable composite (e.g., G-anal Flo X GRADIA DIRECT Flo) to the cavity floor before the application of everX Posterior.
5. Light Curing (Fig. 4)
Light cure everX Posterior using a light curing unit. Keep light guide as close as possible to the composite. Refer to the following chart for Irradiation Time and Effective Depth of Cure.

Table with 3 columns: Irradiation Time, Depth of cure, and Curving depth. Rows include High Power LED Light (>1200 mW/cm²), Halogen/ Normal LED Light (>700 mW/cm²).

SHADE
everX Posterior is available in one universal shade.

STORAGE
Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F) away from high temperatures or direct sunlight.

PACKAGES
15 units (0.25 g / 0.13 mL)
50 units (0.25 g / 0.13 mL)

CAUTION
1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.

CLEANING AND DESINFECTING:
MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

For the Summary of Safety and Clinical Performance (SSCP) please see EUADMED database (https://ec.europa.eu/tools/edumed/) or contact us at Regulatory.gcc@dent.nl

Indesired effects & Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by you or third parties, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the product authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

UK Responsible Person:
GCO IP B.V. Industrieweg 14D, Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JL, United Kingdom

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Removal of everX Posterior:
Remove the cap, and extrude everX Posterior material directly into the cavity. Pack the everX Posterior in the cavity, using suitable instruments, until it has been adapted tightly against the cavity walls. To prevent premature curing, avoid bright light in the work area. When placing everX Posterior material directly into the cavity, ensure that there is enough space for the overlying composite on all surfaces - and of large cavities (Fig. 3a, 3b).

- 5. Light Curing (Fig. 4)
Optional, apply first a thin layer of flowable composite (e.g., G-anal Flo X GRADIA DIRECT Flo) to the cavity floor before the application of everX Posterior. Place the everX Posterior on top of the flowable composite and pack it in the cavity.

SHADE
everX Posterior is available in one universal shade.

STORAGE
Recommended for optimal performance, store in a cool and dark place (4-25°C / 39-77°F) away from high temperatures or direct sunlight.

PACKAGES
15 units (0.25 g / 0.13 mL)
50 units (0.25 g / 0.13 mL)

CAUTION
1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water. To avoid contact, a rubber dam and/or COCOA BUTTER can be used to isolate the operation field from the skin or oral tissue.

CLEANING AND DESINFECTING:
MULTI-USE DELIVERY SYSTEMS: To avoid cross-contamination between patients this device requires mid-level disinfection. Immediately after use inspect device and label for deterioration. Discard device if damaged.

For the Summary of Safety and Clinical Performance (SSCP) please see EUADMED database (https://ec.europa.eu/tools/edumed/) or contact us at Regulatory.gcc@dent.nl

Indesired effects & Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by you or third parties, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the product authority of your country accessible via the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en

UK Responsible Person:
GCO IP B.V. Industrieweg 14D, Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JL, United Kingdom

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