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Instruktion for Use for REFILLS

#### GRADIA™ PLUS MODULAR COMPOSITE SYSTEM FOR INDIRECT RESTORATIONS

For use only by a dental professional in the indications for use.

#### INDICATIONS FOR USE

- Crown and Bridge-work (with metal backing).
- Metal free restorations: Jacket Crowns, Inlays, Onlays and Laminated Veneers.
- Implant superstructures.
- Reproduction of gum tissue for crown restorations.

#### CONTRAINDICATIONS

Avoid use of this product in patients with known allergies to methacrylate monomer, methacrylate polymer or alcohol.

#### PRODUCT DESCRIPTION

GRADIA PLUS is a light-cure material for crown and bridgeworks including occlusal surfaces. The particle size of inorganic fillers range is 0.02-1.0 µm for OPAQUE, GUM SHADES OPAQUE, 0.01-5.5 µm for PASTE HB, PASTE LB, GUM SHADES HB, GUM SHADES LB.

The total amount of inorganic filler is approximately 51-75 wt% for OPAQUE, GUM SHADES OPAQUE, 66-80 wt% for PASTE HB, PASTE LB, GUM SHADES HB, GUM SHADES LB.

#### COMPOSITION

OPAQUE, PASTE HB, PASTE LB, GUM Barium glass, dimethacrylate, silicon dioxide, initiator, pigment, stabilizer
LUSTRE PAINT Multifunctional monomer, initiator, methacrylate, silicon dioxide, pigment
GRADIA PLUS Lustre Paint Diluting Liquid Methacrylate
GRADIA PLUS Modeling Liquid Dimethacrylate, silicon dioxide, stabilizer, initiator

#### IRRADIATION TIME AND CURING UNIT

Curing unit	LABOUGHT DUO		STEP-LIGHT SL-I	LABO-LIGHT LV/III/II
	Step-mode**	Full-mode	Pre-Cure***	Final cure
OPAQUE	-	1 min.	-	1 min.
PASTE HB, PASTE LB, GUM SHADES LB, GUM SHADES HB	10 sec.*	3 min.	10 sec.*	3 min.
LUSTRE PAINT***	10 sec.*	90 sec.	10 sec.*	5 min.

\* For one surface of a single crown.
\*\* Distance from light source: 3cm.
\*\*\* Thickness: 0.1mm or less.

The Color Chart of GRADIA PLUS is shown in the manual that is available at <http://www.gecorpoe.com> or for The Americas <http://www.gcamerica.com>

#### CAUTION

- In rare cases, this product may cause sensitivity in some people. If you experience reactions or are experienced, discontinue the use of the product and consult a physician.
- Wear eye protection and plastic or rubber gloves during operation in particular to avoid direct contact with paste and air inhibited resin layers in order to prevent possible sensitivity. LUSTRE PAINT DILUTING LIQUID, SEPARATOR, DIE-HARDNER, METAL PRIMER II / GC, METAL PRIMER Z and GC CERAMIC PRIMER II are highly flammable liquid and vapour. Do not use near naked flame. Keep away from sources of ignition. Keep away from direct sunlight.
- After dispensing, use immediately and replace caps on syringe or bottle.
- When adjusting or polishing the veneer surface, use a dust collector and wear a dust mask to avoid inhaling dust.
- In case of contact with oral tissues or skin, immediately remove with cotton. Flush with water.
- In case of contact with eyes, immediately wash the eye with copious amounts of water and seek medical attention.
- This product should not be used for malocclusion, clenching or bruxism.
- At the time of cementation, sandblast the inner surface of metal free restorations, and then be sure to treat the inner surface with a silane coupling agent (e.g. CERAMIC PRIMER II (GC)).
- Dispose all waste according to local regulation.
- To prevent cross contamination between patients, do not reuse the dispensing tip.
- The dispensing tip cannot be sterilized in an autoclave or chemically.
- Wear protective eye glasses during light curing.
- Lower light intensity may cause insufficient curing.

#### STORAGE

Recommended for optimal performance, store in a cool place (4-25°C / 39-27.0°F) away from high temperatures or direct sunlight.

Some products referenced in the present IFU may be classified as hazardous according to GHS.

Always familiarize yourself with the safety data sheets available at: <http://www.gecorpoe.com>

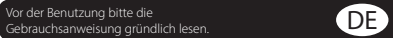
They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SCSP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed/>) or contact us at [Regulatory.gce@gc.dental](mailto:Regulatory.gce@gc.dental)

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)
In this way you will contribute to improve the safety of this product.

**UK Responsible person**
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell
Buckinghamshire
MK16 8JS
United Kingdom

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Unwanted effects reports:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental)
In this way you will contribute to improve the safety of this product.

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#### PRODUCT DESCRIPTION

GRADIA PLUS is a light-cure material for crown and bridgeworks including occlusal surfaces. The particle size of inorganic fillers range is 0.02-1.0 µm for OPAQUE, GUM SHADES OPAQUE, 0.01-5.5 µm for PASTE HB, PASTE LB, GUM SHADES HB, GUM SHADES LB. The total amount of inorganic filler is approximately 51-75 wt% for OPAQUE, GUM SHADES OPAQUE, 66-80 wt% for PASTE HB, PASTE LB, GUM SHADES HB, GUM SHADES LB.

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