# Master all your challenges

*initial*<sup>™</sup> Lustre Pastes NF Gum Shades


- EN Technical Manual
- DE Technische Arbeitsanleitung
- FR Manuel Technique
- IT Manuale tecnico
- ES Manual técnico
- PT Manual técnico

MAY 2024

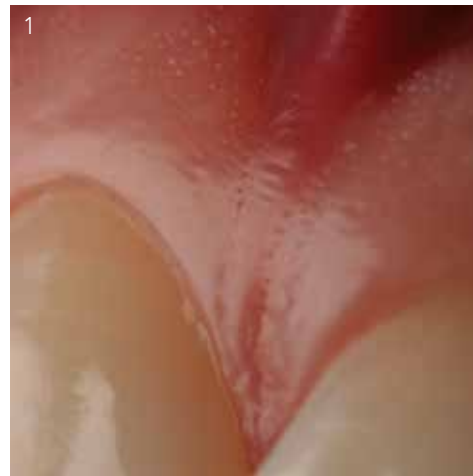- EN True-to-nature ...
- DE Naturidentisch ...
- FR Proche du naturel ...
- IT Estetica naturale ...

Photoshooting 1 - 4: CDT D. Ibraimi, Switzerland  
Photoshooting 5 - 6: Byrnes Dental Laboratory, UK

EN	DE	FR	IT	Page
Table of Contents	Inhaltsverzeichnis	Table des matières	Indice	
Intended Use	Bestimmungsgemäßer Gebrauch	Utilisation	Campo di impiego	4-5
Description of the Lustre Pastes NF Gum Shades	Beschreibung der Lustre Pastes NF Gum Shades	Description des Lustre Pastes NF Gum Shades	Descrizione delle Lustre Pastes NF Gum Shades	6-7
Basic colours	Basis-Farben	Teintes de base	Colori di base	
Effect colours	Effekt-Farben	Effets de teinte	Colori per effetti	
The liquids of the Lustre Pastes NF Gum Shades Set	Die Flüssigkeiten des Lustre Pastes NF Gum Shades Set	Les liquides du coffret Lustre Paste NF Gum Shades	I liquidi del set di Lustre Paste NF Gum Shades	8-9
Lustre Pastes NF Gum Shades Application	Lustre Pastes NF Gum Shades Anwendung	Application Lustre Pastes NF Gum Shades	Applicazione delle Lustre Pastes NF Gum Shades	
Basic Step by Step Instruction (Zirconium Frame)	Basis-Farben Step by Step Anleitung (Zirkon-Gerüst)	Etape par Etape (armature zircon)	Istruzioni fondamentali passo per passo (frame in zirconia)	10-15
Basic Instruction (Non Precious & Ti Dental Frame)	Basis-Farben Anleitung (Nichtedelmetall- / Ti-Gerüst)	Etape par Etape (armature dentaire non précieuse / armature Titane)	Istruzioni di base (frame in leg vile / struttura in Titanio)	16-17
Firing Chart	Brenntabelle	Tableau des températures	Grafico delle cotture	18
Tips and Tricks	Tipps und Tricks	Trucs et astuces	Consigli e accorgimenti	19
Facilities of the Lustre Pastes NF Gum Shades	Möglichkeiten mit Lustre Pastes NF Gum Shades	Exemples d'application des les Lustre Pastes NF Gum Shades	Esempi di applicazioni delle Lustre Pastes NF Gum Shades	20-21
Physical Properties & Storage	Physikalische Eigenschaften und Lagerung	Propriétés physiques & Conservation	Proprietà fisiche e Conservazione	22
FAQ	FAQ	FAQ	Domande più frequenti	23-24
Medical Device Regulation Information	Informationen zur Medizinprodukteverordnung	Informations sur la réglementation des dispositifs médicaux	Informazioni sul regolamento dei dispositivi medici	51-52

# GC Initial IQ Lustre Pastes NF Gum Shades

## EN Intended Use

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### EN Paint-on your gum shades

GC Initial IQ Lustre Pastes NF Gum Shades are 3-dimensional ceramic stains developed to enable dental technicians to create a lifelike transition from the crown margin to the gingiva just by painting the gum shades. With a continuously growing demand for aesthetic implant-supported superstructures, there has also been a growing desire among dental technicians for a ceramic staining approach for the soft tissue zones. With the new unique Lustre Pastes NF Gum Shades Set, you can from now on bring not only the white, but also the red aesthetics of your crowns and bridges to life. These unique gum coloured Lustre Pastes allow an individual adaptation to match natural gingival shades offering a natural harmony. With their optimal match to both "low" and "high" CTE ceramics, the system is compatible with the complete GC Initial ceramic range. Available in a ready to use consistency, the Lustre Pastes NF Gum Shades are based on fine ceramic particles and can be applied in a thicker layer where they will exhibit unsurpassed nature analogue gum reproduction and a very natural glaze to your restorations.

### Features

- One kit for the complete GC Initial ceramic range.
- No complex layering or extra firing cycles required. Fast – Easy – Secure.
- Available in ready to use jars for a more convenient application.
- Diluting and Refresh liquids to keep the optimal consistency until the end.
- Five gum coloured pastes with an ideal thixotropy for easy and precise application.
- Four mixable Gum Modifiers allowing unlimited creativity to copy natural gingival harmony.

### Indications

The Lustre Pastes NF Gum Shades are a one for all solution for the complete Initial line from GC. It's possible to use the Lustre Pastes NF Gum Shades in a wide CTE – Range in order to work with nearly all types of ceramic. These Lustre Pastes NF Gum Shades represent a modern alternative for conventional staining of gingival areas. Due to the unique properties it's just a question of minutes to individualize the gingival areas of your crown and bridge work.

**Keep in mind: What you paint (on) is what you get (after firing)**

### Contraindications

Individualization of restorations in the non-approved CTE area (CTE = coefficient of thermal expansion of the GC Initial IQ ONE SQIN Concept. The respective CTE must be compatible).

### Warnings

Inhalation of ceramic dust released during processing must be avoided by using suitable suction devices and, if necessary, protective masks. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

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LP-M2  
Modifier Red

**EN Indications:**

The modifiers can be used to intensify the single Lustre Pastes Gum Shades allowing an unlimited reproduction of gum colours.

**Instructions for use:**

The different modifiers can be added & mixed directly with the Lustre Pastes Gum Shades. The desired colour can visually be controlled.

When adding modifiers to the Lustre Pastes Gum Shades there is no need to adapt the firing parameters.

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#### EN Lustre Paste 'Diluting Liquid':

The 'Diluting Liquid' is a special liquid to change the flowability of the Lustre Pastes. However, it is recommended to obtain a gel/paste like consistency of the Lustre Pastes for best results.

Attention:

- If the Lustre Pastes are too much diluted, the result is a reduction in the gloss of the paste after the firing; hence, the desired result may not be achieved in one glaze firing.
- When needed, dilute the desired amounts of the pastes in a separate plate with Diluting Liquid, never dilute directly in the jars.

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#### EN **Lustre Paste 'Refreshing Liquid':**

The 'Refreshing Liquid' is used to preserve the thixotropic property of the pastes. This liquid is used to recover dried out pastes to an optimum, gel like consistency without changing the physical properties. The Refresh Liquid can be used both in the jars and mixed on a separate plate.

Important: Please shake the Refresh Liquid well before use!

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EN After the second firing the restoration can be shaped & contoured in the usual way. Before applying the Lustre Coating, the ceramic restoration is shaped and contoured using adjusted diamond burs. After polishing with a ceramic polisher the finished Crowns & Bridges are sand-blasted with 50 microns Alu-Oxid at 1,5 bar pressure.

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EN To intensify the areas between the tooth roots, e.g. Lustre Paste G-36 can be used.

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EN For more detailed instructions, please refer to the dedicated Technical Manuals / IFU of each of the below GC Initial Lines:

- GC Initial MC/LF
- GC Initial Ti
- GC Initial Zr-FS
- GC Initial LiSi
- GC Initial IQ - SQIN
- GC Initial Ti Bonder
- GC Initial IQ Layering-over-Metal
- GC Initial IQ Press-over-Metal
- GC Initial INmetalbond

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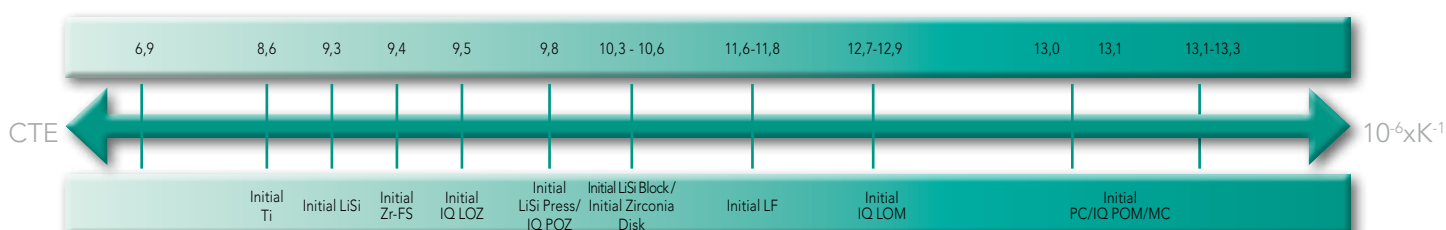
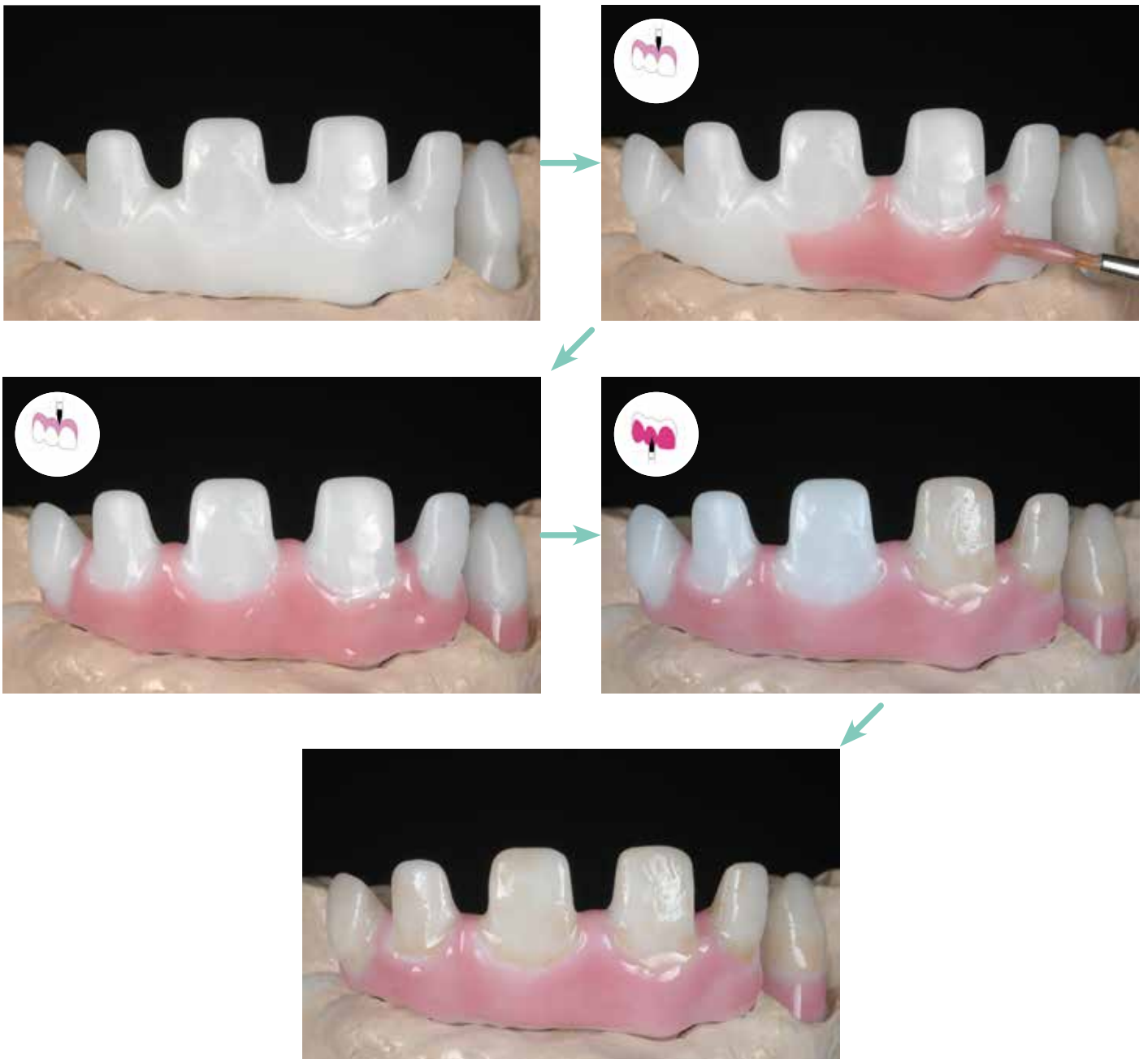
**Classification: Dental Ceramics Type I / Class 1**

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## Storage / Lagerung / Conservation / Conservazione

Recommended for optimal performance, store at 4-25°C (39-77°F).

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1x Base Light, 1x Base Dark, 1x Intensive Violet, 1x Intensive Red,  
1x Intensive Cream, 4x Modifier (white, red, blue, brown),  
1x Diluting Liquid, 1x Refresh Liquid, 1 mixing dish,  
2 brushes, 1x Technical Manual, 1x Colour Chart

EN FAQ

DE FAQ

FR FAQ

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## EN Basic Rule: What you paint (on) is what you get (after firing)

### Q1. How can I dilute (adapt the consistency of) the Lustre Pastes NF Gum Shades?

Use the Lustre Paste ONE/NF Diluting Liquid to adapt the consistency.

Do NOT use water to dilute. LP may not come in contact with water (see page 8).

### Q2. What can I do when the consistency of the Lustre Pastes NF Gum Shades is too dry?

Use the Refresh Liquid to recover the Lustre Pastes in case of dryness (see page 9).

### Q3. How do we change the texture of the Lustre Pastes NF Gum Shades?

The requested surface texture/smoothness of the Lustre Pastes NF Gum Shades layer can be changed by soft vibration or condensing the restoration.

### Q4. I do not have a shiny appearance after firing the Lustre Pastes NF Gum Shades, what is the cause?

Check the firing temperatures, increase the final temperature. When the Lustre Pastes NF Gum Shades are heavily diluted or applied in a too thin layer the glossy effect is tempered, so apply a more thick layer/dilute less the pastes.

### Q5. Can I apply more than one layer of Lustre Pastes NF Gum Shades?

Yes, you can do a second firing, keeping the same firing temperatures.

### Q6. Can I mix the Lustre Pastes NF Gum Shades with other Initial powders?

You can use the GC Initial Modifier powders to intensify the Lustre Pastes NF Gum Shades.

Mix the required amount of Modifier powder with the Lustre Pastes ONE/NF Diluting Liquid and mix it with the required LP or directly mix the Modifier powders with e.g. the Lustre Pastes NF Gum Shades.

### Q7. Are the Lustre Pastes NF Gum Shades sufficient abrasion resistant?

Yes, the LP NF Gum Shades are a special mixture of different types of low fusing ceramic particles, stains & glaze mixture offering lifelike aesthetics.

### Q8. The glaze firing is blarily, spotted or cracked?

The Lustre Pastes NF Gum Shades came partly or completely in contact with water.

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GC Initial IQ SQIN, Shades Set

# GC Initial IQ Lustre Pastes NF Gum Shades

## EN Medical Device Regulation Information

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#### Liquids:

H<sub>2</sub>O + Propylenglykol

- EN Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas: <http://www.gcamerica.com> They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at [Regulatory.gce@gc.dental](mailto:Regulatory.gce@gc.dental).

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: [https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en) as well as to our internal vigilance system: [vigilance@gc.dental](mailto:vigilance@gc.dental). In this way you will contribute to improve the safety of this product.

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GC NORDIC AB - Finnish Branch
GC NORDIC AB - GC Nordic Danish Branch
GC NORDIC AB
GC AUSTRIA GmbH - Swiss Office
GC AUSTRALASIA DENTAL PTY LTD
GC ASIA DENTAL PTE. LTD.