Master all your challenges

initial[™]
Spectrum Stains

- EN Technical Manual
- DE Technisches Handbuch
- FR Manuel Technique
- IT Manuale tecnico
- ES Manual técnico
- PL Instrukcja techniczna

JANUARY 2023

GC

E	DE	FR	IT	
Table of Contents	Inhaltsverzeichnis	Table des matières	Indice	Page
Intended Use	Bestimmungsgemäße Verwendung	Indications	Destinazione d'uso	4 - 5
Description of GC Initial Spectrum Stains	Beschreibung der GC Initial Spectrum Stains	Description des GC Initial Spectrum Stains	Descrizione di GC Initial Spectrum Stains	6
The Glaze & Glaze Liquids	Glasurflüssigkeit und Glasurpaste	Poudre Glaze & Liquides Glaze	La polvere e i liquidi per glasura	7
GC Initial Spectrum Stain Application	Anwendung der GC Initial Spectrum Stains	Application des GC Initial Spectrum Stains	Applicazione di GC Initial Spectrum Stain	
External Characterization	Äußere Charakterisierung	Caractérisation externe	Caratterizzazione esterna	8 - 11
Internal Characterization	Innere Charakterisierung	Caractérisation interne	Caratterizzazione interna	12
Combined Characterization	Kombinierte Charakterisierung	Caractérisation combinée	Caratterizzazione combinata	14 - 15
Firing Chart	Brenntabelle	Tableau de cuisson	Tabella dei parametri di cottura	16
Physical Properties & Storage	Physikalische Eigenschaften und Lagerung	Propriétés physiques & Conservation	Proprietà fisiche e Conservazione	17
FAQ	Häufig gestellte Fragen	FAQ	Domande frequenti	18 - 22
Related Products	Verwandte Produkte	Produits associés	Prodotti collegati	42
Medical Device Regulation Information	Informationen zur Medizinprodukteverordnung	Informations sur la réglementation des dispositifs médicaux	Informazioni sul regolamento dei dispositivi medici	43-44

EN Intended Use

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FR Indications

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EN One for All

GC Initial Spectrum Stains are universal stains & glaze for the individualized staining and characterization of ceramic materials.

• A wide spectrum of use

The stains and glaze are developed for the external staining and glazing of tooth coloured ceramic materials and internal individualization of layering ceramics. Compatible to a wide CTE-range, the GC Initial Spectrum Stains are perfectly adapted to fit on nearly all types of dental ceramics.

In particular they are adapted to the aesthetical colour concept of the GC Initial ceramic portfolio.

• A wide spectrum of colours

GC Initial Spectrum Stains are powder stains available in a variety of 20 colours empowering the aesthetics of your restorations.

• Adaptable consistency of the stains & glaze

By using either the Glaze or the Glaze Paste Liquid the stains & glaze can be adapted to the preferred consistency of the user.

Regardless the way of application a very fine end result will be obtained maintaining the surface texture and assuring an optimal gloss level.

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-  Initial Spectrum Stain SPS-1, 3g - ivory white

 Initial Spectrum Stain SPS-2, 3g - melon yellow

 Initial Spectrum Stain SPS-3, 3g - signal yellow

 Initial Spectrum Stain SPS-4, 3g - light terracotta

 Initial Spectrum Stain SPS-5, 3g - savannah gold

 Initial Spectrum Stain SPS-6, 3g - caramel

 Initial Spectrum Stain SPS-7, 3g - camel

 Initial Spectrum Stain SPS-8, 3g - olive

 Initial Spectrum Stain SPS-9, 3g - brown

 Initial Spectrum Stain SPS-10, 3g - dark walnut
-  Initial Spectrum Stain SPS-11, 3g - coral

 Initial Spectrum Stain SPS-12, 3g - denim blue

 Initial Spectrum Stain SPS-13, 3g - twilight

 Initial Spectrum Stain SPS-14, 3g - deep orange

 Initial Spectrum Stain SPS-15, 3g - sunset

 Initial Spectrum Stain SPS-16, 3g - midnight

 Initial Spectrum Stain SPS-17, 3g - dark brown

 Initial Spectrum Stain SPS-18, 3g - illusion 1

 Initial Spectrum Stain SPS-19, 3g - illusion 2

 Initial Spectrum Stain SPS-20, 3g - illusion 3

-  SPS-17
-  SPS-18
-  SPS-19
-  SPS-20

EN The Glaze & Glaze Liquids
 DE Glasurflüssigkeit und Glasurpaste
 FR Poudre & Liquides Glaze
 IT La polvere e i liquidi per glasura

EN **GC Initial Spectrum Glaze Powder GL, 10g**

Fine Glaze powder to be mixed with the Glaze or Glaze Paste Liquid.

The Glaze Powder can be adapted to the preferred consistency of the user by using the Glaze or the Glaze Paste Liquid.

GC Initial Spectrum Glaze Powder GL-FLUO, 10g

Fine fluorescent Glaze powder to be mixed with the Glaze or Glaze Paste Liquid.

The fluorescent Glaze Powder can be adapted to the preferred consistency of the user by using the Glaze or the Glaze Paste Liquid.

GC Initial Spectrum Glaze Liquid, 25ml

The standard low viscosity type of mixing liquid allowing a fine application.

GC Initial Spectrum Glaze Paste Liquid, 8ml

The high viscosity type of mixing liquid allowing a gel-type application.

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GC Initial Spectrum Liquide Glaze, 25ml

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IT **GC Initial Spectrum Glaze Powder GL, 10g**

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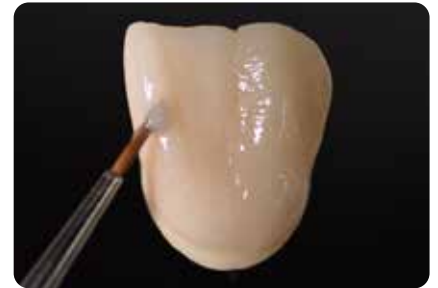
EN Initial situation

Ceramic crown with individual micro-texture, ready to be stained & glazed. *Note: before staining and glazing, the ceramic needs to be sand-blasted with 50 microns Alu-Oxid at 1,5 bar pressure and steam cleaned or cleaned by ultra-sonic bath.*

By using either the Glaze or the Glaze Paste Liquid the stains can be adapted to the preferred consistency.

Left: "Gel-type" mixture with Glaze Paste Liquid.

Right: "Fine" mixture with Glaze Liquid.

Coat the entire surface of the restoration with the Spectrum Glaze in the preferred consistency.

Left part of the crown: application of the Glaze using the Glaze Paste Liquid.

Right part of the crown: Glaze applied using the Glaze and Glaze Liquid.

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EN Initial LiSi Press Crown
As connection firing Lustre Pastes ONE have been used on the entire surface.
For extra internal individualization the Initial Spectrum Stains are applied and fired.

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EN Layering of the Initial LiSi Press Crown with the Initial LiSi veneering ceramic.
Initial Spectrum Stains are used for internal individualization during the layering process.

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EN Please consult the firing parameters of the dedicated layering materials.

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EN External staining of the Initial Lisi Press Crown to mimic nature analogue ageing of the tooth.

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EN Final result after internal and external staining.

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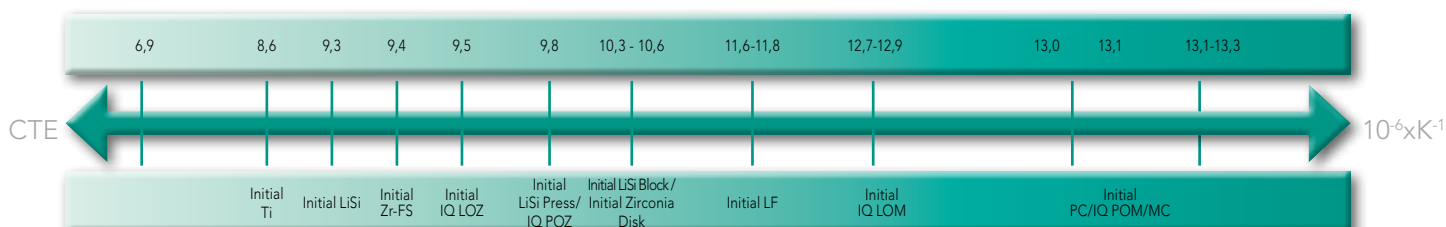
Classification: Dental Ceramics Type I / Class 1

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Storage / Lagerung / Conservation / Conservazione

Recommended for optimal performance, store at 4-25°C (39-77°F).

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GC Initial Spectrum Stains Set

Content / Inhalt / Contenu / Contenuti

- 16x GC Initial Spectrum Stain SPS-1 - SPS-16, 3g
- 1x GC Initial Spectrum Glaze Powder GL, 10g
- 1x GC Initial Spectrum Glaze Liquid, 25ml
- 1x GC Initial Spectrum Glaze Paste Liquid, 8ml
- 1x GC Initial Spectrum Stain Shade Guide

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- GC Initial Spectrum Stain SPS-17 - SPS-20, 3g
- GC Initial Spectrum Glaze Powder GL-FLUO, 10g

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EN Important to know:

- GC Initial LRF Block
It is advised to use the GC Initial LRF Glaze Paste as dedicated glazing material.
The GC Initial Spectrum Stains can be added in limited amounts to the GC Initial LRF Glaze Paste for individualisation.
- Layering materials can be individually toned using the Spectrum Stains.
When mixed into Dentin, Enamel, Transpa powders it is advised to use small amounts.
To assure the right mixing ratio we advise to create a small firing sample.
- When used during build-up on an unfired ceramic small amounts can be added.
- When mixing layering material and stains use the recommended modelling liquid not the Initial Spectrum Stain Liquid.
- Always consult the firing parameters of the dedicated layering materials.

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EN Q2. Are the GC Initial Spectrum Stains & Glaze fluorescent?

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EN Similar to natural teeth the GC Initial Spectrum Stains offer a certain level of fluorescence.

The Glaze Powder GL-FLUO however is a highly fluorescent glazing material.

It protects and glazes your surface.

Fluorescence is also given by the underlying ceramic powders / core material.

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EN Q3. Can the GC Initial Spectrum Stains be mixed with the GC Initial IQ Lustre Pastes ONE?

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EN Yes they can be mixed with moderation to intensify the Lustre Pastes ONE.

Mix the required amount of stain powder with the LP NF/ONE Dilution Liquid and mix it with the required LP ONE or directly mix the stain powders with e.g. the Lustre Paste Neutral Fluo (L-NFL).

Initial Spectrum Stains and Lustre Pastes ONE are compatible products.

Initial Lustre Pastes ONE are ready-to-use 3-dimensional dynamic shades and are based on a special mix of fine ceramic particles.

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-  Initial Spectrum Stain SPS-1, 3g - ivory white
-  Initial Spectrum Stain SPS-2, 3g - melon yellow
-  Initial Spectrum Stain SPS-3, 3g - signal yellow
-  Initial Spectrum Stain SPS-4, 3g - light terracotta
-  Initial Spectrum Stain SPS-5, 3g - savannah gold
-  Initial Spectrum Stain SPS-6, 3g - caramel
-  Initial Spectrum Stain SPS-7, 3g - camel
-  Initial Spectrum Stain SPS-8, 3g - olive
-  Initial Spectrum Stain SPS-9, 3g - brown
-  Initial Spectrum Stain SPS-10, 3g - dark walnut
-  Initial Spectrum Stain SPS-11, 3g - coral
-  Initial Spectrum Stain SPS-12, 3g - denim blue
-  Initial Spectrum Stain SPS-13, 3g - twilight
-  Initial Spectrum Stain SPS-14, 3g - deep orange
-  Initial Spectrum Stain SPS-15, 3g - sunset
-  Initial Spectrum Stain SPS-16, 3g - midnight
-  Initial Spectrum Stain SPS-17, 3g - dark brown
-  Initial Spectrum Stain SPS-18, 3g - illusion 1
-  Initial Spectrum Stain SPS-19, 3g - illusion 2
-  Initial Spectrum Stain SPS-20, 3g - illusion 3

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GC Initial MC, Zr-FS, LiSi

GC Initial Spectrum Stains

EN Medical Device Regulation Information

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DE

Powders:

SiO₂ 61-65%, Al₂O₃ 8-10%, K₂O 5-7%, Na₂O 7-10%, CaO 1-2%, BaO 2-3%,
B₂O₃ 7-10%, TiO₂ <1%, ZrO₂ <1%, P₂O₅ <1%, CeO₂-CeF₃ <1%, Pigments 0-3%

Liquids:

H₂O + Propylenglykol

EN Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: <http://www.gceurope.com> or for The Americas: <http://www.gcamerica.com> They can also be obtained from your supplier.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental.

Undesired effects - Reporting: If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

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GC NORDIC AB - Finnish Branch
GC NORDIC AB - GC Nordic Danish Branch
GC NORDIC AB - c/o Lundin Revisionbyrå
GC AUSTRIA GmbH - Swiss Office
GC AUSTRALASIA DENTAL PTY LTD
GC ASIA DENTAL PTE. LTD.