

Fit and comfort for an active life



RELINE™ II Soft &
RELINE™ II Extra Soft
from GC

Soft silicone relining materials
in cartridge delivery

GC

Fit, function and comfort to keep on smiling

Irritation, extractions or implant placement will lead to changes in soft and hard tissues and **it will be needed to adapt the dentures to those new local conditions.**

Because elderly patients are getting healthier and stay active longer, they want to be able to continue smiling, speaking and enjoying eating at all times; even when wearing dentures.

Reline II Soft and Extra Soft from GC have been designed to create a soft layer between the denture and the mucosa which will **support healing, maintain comfort and function** for weeks and up to several months.



Pictures courtesy of Dr D. Garcia-Baeza, Spain

An easy-to-apply silicon liner in cartridge delivery

The material's **ideal viscosity** makes it **flow under pressure, adapting nicely** to all anatomical details.

Its **improved trimming ability** makes the ultimate finishing of margins even easier.



Improved trimming ability
for a better finition of margins

Long lasting comfort and gentle relief for your patients

With RELINE II, temperature will not increase during setting, resulting in more comfort for your patients.

The material **remains smooth and odorless** and maintains its **dimensional stability and elasticity** for weeks or months. The result? A **durable & convenient solution** for denture patients.



Protection of margins with Reline II
finishing material

Strong adhesion, yet easy to maintain and adapt

The enhanced composition of **GC RELINE II Primer for resin** provides **strong adhesion to the denture** for a longer time. However, this strong adhesion does not make it any less flexible.

You can **add material within 3 months** after the first application, to adapt to changes of the mucosa and bone. The new **GC RELINE II Remover for resin** makes it possible to remove the relining layer and replace it very quickly.



**GC RELINE II
Primer
for resin**




**GC RELINE II
Remover
for resin**

in all circumstances

From resilience to elasticity:
choose the product that best fits the clinical situation

Featuring a Hardness of 32 (Shore A), **GC RELINE™ II Extra Soft** is a resilient material. It's particularly helpful to **normalise** irritated tissues and **support tissue healing** after extraction or implant surgery. It can stay in place for **days or weeks**.

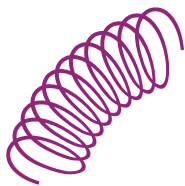

With a Shore A Hardness of 48, **GC RELINE™ II Soft** will maintain its elasticity in time and, much **like a cushion**, **spreads the load pressure** in case of anatomical problems. It is also ideal to reline **implant-supported dentures** after the healing phase. It can stay in place for **weeks or months**.



Straightforward application & handling, throughout the entire procedure



Initial denture, to be relined following the placement of 4 implants.



Relieve the area to be relined and roughen the surface.



Apply GC RELINE II Primer on the area to be relined. Gently dry with air.



Easy cartridge delivery.



Apply GC RELINE II (Extra) Soft. Working time is 2'00".



Seat the denture in the mouth and muscle trim. Retain in situ for 5'00".



Remove the excesses using a scalpel or scissors. Trim with GC RELINE II POINT FOR TRIMMING.



Finish with GC RELINE II WHEEL FOR FINISHING.



Reline denture.

RELINE™ II Soft & RELINE™ II Extra Soft

Fit and comfort for an active life

Discover the all-round family of reline materials from GC



	GC TISSUE CONDITIONER	GC RELINE II Extra Soft	GC RELINE II Soft	GC RELINE P/L
Material	Acrylic Resin	A-Silicone	A-Silicone	Acrylic Resin (MMA Free)
Indications	<ul style="list-style-type: none"> • Pressure points • During implant procedures • Soft relining • Tissue recovery after trauma/surgery • Functional impressions 	<ul style="list-style-type: none"> • Tissue toning material • Sensitive mucosa • Pressure points • Normalising irritated tissues 	<ul style="list-style-type: none"> • Implant cases • Pressure points • Bony undercuts • Pain reduction & increase of comfort • Long-lasting relining in case of missing supportive tissue 	<ul style="list-style-type: none"> • Long term relining • Non fitting dentures without any tissue trauma • Final step after pre-treatment with soft relining materials and/or tissue conditioners
Time in mouth	Days/weeks/month	Days/weeks	Weeks/months	Months/years

Available packages of GC RELINE II Soft & Extra Soft

Intro Kit

010259 **GC RELINE II Soft Intro Kit:**
 1x GC RELINE II Soft Cartridge
 1x GC RELINE II Primer for Resin
 1x GC RELINE II Point for Trimming
 1x GC RELINE II Wheel for finishing
 1x GC RELINE II Remover for Resin

010262 **GC RELINE II Extra Soft Intro Kit:**
 1x GC RELINE II Extra Soft Cartridge
 1x GC RELINE II Primer for Resin
 1x GC RELINE II Point for Trimming
 1x GC RELINE II Wheel for finishing
 1x GC RELINE II Remover for Resin

Refills

010265 GC RELINE II Soft Cartridge, 48 ml
 010268 GC RELINE II Extra Soft Cartridge, 48 ml
 010273 GC RELINE II Primer for Resin, 10 ml
 010271 GC RELINE II Finishing Material (Modifier), 1-1 Pack, 2x 9ml
 010275 GC RELINE II Remover for Resin, 5,5ml
 010277 GC RELINE II Point for Trimming x 3,
 010280 GC RELINE II Wheel for Finishing x 12,
 010283 GC RELINE II Disposable Brush x 6

GC EUROPE N.V.
 Head Office
 Researchpark
 Haasrode-Leuven 1240
 Interleuvenlaan 33
 B-3001 Leuven
 Tel. +32.16.74.10.00
 Fax. +32.16.40.48.32
 info@gceurope.com
 http://www.gceurope.com

GC UNITED KINGDOM Ltd.
 12-15, Coopers Court
 Newport Pagnell
 UK-Bucks. MK16 8JS
 Tel. +44.1908.218.999
 Fax. +44.1908.218.900
 info@uk.gceurope.com
 http://uk.gceurope.com

