Prior to use, carefully read  
the instructions for use.  
EN

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## CARTRIDGE DISPENSER II MIXING TIP II

For use only by a dental professional in the recommended indications.

CARTRIDGE DISPENSER II and MIXING TIP II are used to mix and dispense the required amount of cartridge type vinyl/polyether silicone (EX'Alence) and vinyl/polysiloxane impression and reline materials (particularly GC RELINE® II Soft / Extra Soft).

### RECOMMENDED INDICATIONS

Dispensing GC vinyl/polyether silicone or vinyl/polysiloxane impression and reline materials from cartridges (the Cartridge Dispenser II is referred to as "dispenser" hereafter).

### DIRECTIONS FOR USE

1. Lift the release lever of the dispenser and pull the piston plunger all the way back into the dispenser.
2. Lift the cartridge holder of the dispenser and load the cartridge ensuring that the V-shaped notch on the flange of the cartridge is facing down. Push down the cartridge holder to fix the cartridge.
3. Lift the release lever and push the piston plunger forward until it engages into the cartridge.
4. Remove the cartridge cap by rotating 1/4 turn anti-clockwise, tilt the cap downward and peel it away from the cartridge. Gently squeeze the dispenser handle to extrude a small amount of material from the two openings at the end of the cartridge. Make sure that base and catalyst come out evenly.
5. Align the V-shaped notch on the rim of MIXING TIP II to the V-shaped notch between the cartridge barrels. Push firmly to attach the mixing tip. Then, rotate the coloured collar of the mixing tip 1/4 turn clockwise to the end of the cartridge. The dispenser is now ready for use.
6. Squeeze the handle several times to extrude the material. After use, do not remove the MIXING TIP II as this will become the storage cap until next use. When replacing the MIXING TIP II, rotate the collar on the mixing tip 1/4 turn anti-clockwise to align the V-shaped notch on the cartridge. Tilt the mixing tip downward and peel it away from the cartridge.
7. Remove and replace the old mixing tip immediately prior to next use. Before attaching a new tip, gently extrude a small amount of material to ensure that base and catalyst are flowing evenly from both openings. If the materials fail to extrude, remove any hardened materials on the end of the cartridge.
8. To replace the cartridge, lift the release lever and retract the plunger fully. Remove the empty cartridge by lifting the cartridge holder and load a new one into the dispenser.

### IDENTIFICATION OF PARTS

- a) Cartridge Holder
- b) Piston Plunger
- c) Coloured Collar for MIXING TIP II
- d) Release Lever
- e) V-shaped Notch on the flange of the cartridge
- f) V-shaped Notch between the cartridge barrels
- g) V-shaped Notch on the rim of MIXING TIP II

### NOTES

1. Use only after checking that the cartridge and the mixing tip are loaded securely into the dispenser.
2. Sudden extension could cause cracking of the cartridge. To avoid this, gently squeeze the handle to extrude the material.
3. Clean the dispenser with an ethanol solution (for disinfecting) or dental wet tissue. The dispenser must be sterilized in an autoclave. However, with some autoclaves, the temperature of the sterilizing chamber may reach 132 °C or higher and there is a possibility that the dispenser material (synthetic resin) may become distorted at these temperatures. For details, contact the manufacturer of your autoclave.
4. After autoclaving, do not use the dispenser until it has cooled down completely.

### PACKAGES

1. CARTRIDGE DISPENSER II  
1 piece
2. MIXING TIP II  
SS (yellow), 60 pieces
3. MIXING TIP II  
S (pink), 60 pieces
4. MIXING TIP II  
LL (purple), 60 pieces
5. MIXING TIP II  
L (green / blue), 60 pieces

### Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:  
[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

as well as to our internal vigilance system:

vigilance@gc.dental

In this way will contribute to improve the safety of this product.

Last revised : 07/2019

DISTRIBUTED by  
GC CORPORATION

76-1 Hasumura-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

PRINTED IN BELGIUM

## CARTRIDGE DISPENSER II MIXING TIP II

Za upotrebu samo od dentálni specijalisti za preporučenite indikacije.

CARTRIDGE DISPENSER II and MIXING TIP II se izpolzovata za smiesevanje in dозiranje po potrebnem kolicinama vinylo-polyeter silikonovih (EX'Alence) in vinyl-polysiloxanovih otiskovacih in podkladnicah materijala (zejmene GC RELINE® II Soft / Extra Soft).

### RECOMMENDED INDICATIONS

Dispensing GC vinyl/polyether silicone or vinyl/polysiloxane impression and reline materials from cartridges (the Cartridge Dispenser II is referred to as "dispenser" hereafter).

### DIRECTIONS FOR USE

1. Lift the release lever of the dispenser and pull the piston plunger all the way back into the dispenser.
2. Lift the cartridge holder of the dispenser and load the cartridge ensuring that the V-shaped notch on the flange of the cartridge is facing down. Push down the cartridge holder to fix the cartridge.
3. Lift the release lever and push the piston plunger forward until it engages into the cartridge.
4. Remove the cartridge cap by rotating 1/4 turn anti-clockwise, tilt the cap downward and peel it away from the cartridge. Gently squeeze the dispenser handle to extrude a small amount of material from the two openings at the end of the cartridge. Make sure that base and catalyst come out evenly.
5. Align the V-shaped notch on the rim of MIXING TIP II to the V-shaped notch between the cartridge barrels. Push firmly to attach the mixing tip. Then, rotate the coloured collar of the mixing tip 1/4 turn clockwise to the end of the cartridge. The dispenser is now ready for use.
6. Squeeze the handle several times to extrude the material. After use, do not remove the MIXING TIP II as this will become the storage cap until next use. When replacing the MIXING TIP II, rotate the collar on the mixing tip 1/4 turn anti-clockwise to align the V-shaped notch on the cartridge. Tilt the mixing tip downward and peel it away from the cartridge.
7. Remove and replace the old mixing tip immediately prior to next use. Before attaching a new tip, gently extrude a small amount of material to ensure that base and catalyst are flowing evenly from both openings. If the materials fail to extrude, remove any hardened materials on the end of the cartridge.
8. To replace the cartridge, lift the release lever and retract the plunger fully. Remove the empty cartridge by lifting the cartridge holder and load a new one into the dispenser.

### IDENTIFICATION OF PARTS

- a) Cartridge Holder
- b) Piston Plunger
- c) Coloured Collar for MIXING TIP II
- d) Release Lever
- e) V-shaped Notch on the flange of the cartridge

- f) V-shaped Notch between the cartridge barrels

- g) V-shaped Notch on the rim of MIXING TIP II

### NOTES

1. Use only after checking that the cartridge and the mixing tip are loaded securely into the dispenser.

2. Sudden extension could cause cracking of the cartridge. To avoid this, gently squeeze the handle to extrude the material.

3. Clean the dispenser with an ethanol solution (for disinfecting) or dental wet tissue. The dispenser must be sterilized in an autoclave. However, with some autoclaves, the temperature of the sterilizing chamber may reach 132 °C or higher and there is a possibility that the dispenser material (synthetic resin) may become distorted at these temperatures. For details, contact the manufacturer of your autoclave.

4. After autoclaving, do not use the dispenser until it has cooled down completely.

### PACKAGES

1. CARTRIDGE DISPENSER II  
1 piece
2. MIXING TIP II  
SS (yellow), 60 pieces
3. MIXING TIP II  
S (pink), 60 pieces
4. MIXING TIP II  
LL (purple), 60 pieces
5. MIXING TIP II  
L (green / blue), 60 pieces

### Undesired effects- Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link:  
[https://ec.europa.eu/growth/sectors/medical-devices/contacts\\_en](https://ec.europa.eu/growth/sectors/medical-devices/contacts_en)

as well as to our internal vigilance system:

vigilance@gc.dental

In this way will contribute to improve the safety of this product.

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