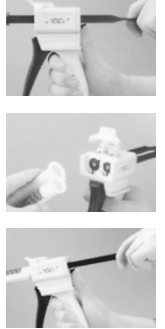
3000157-NT9807
IFU Cartridge Dispenser II EEP

CARTRIDGE DISPENSER II MIXING TIP II

For use only by a dental professional in the indications for use.

CARTRIDGE DISPENSER II and MIXING TIP II are used to mix and dispense the required amount of cartridge type vinyl polyether silicone (EXA'ence) and vinyl polysiloxane impression and relines materials (particularly GCRELIN® II Soft/ExtraSoft)

INDICATIONS FOR USE

Dispensing GC vinyl polyether silicone or vinyl polysiloxane impression and reline materials from cartridge-ges(the Cartridge Dispenser II referreedaa's dispenser hereafter).

DIRECTIONS FOR USE

1. Lift the release lever of the dispenser and pull the piston plunger all the wayback into thedispenser.
2. Lift the cartridge holder of the dispenser and load the cartridge ensuring that the V-shaped notch on the flange of the cartridge is facing down. Push down the cartridge holder to fix thecartridge.
3. Lift the release lever and push the piston plunger forward until it engages into thecartridge.
4. Remove the cartridge cap by rotating 1/4 turn anti-clockwise, tilt the cap downward and peel it away from the cartridge. Gently squeeze the dispenser handle to extrude a small amount of material from the two openings at the end of the cartridge. Make sure that base and catalyst come out evenly.
5. Align the V-shaped notch on the rim of MIXING TIP II to the V-shaped notch between the cartridge barrels. Push firmly to attach the mixing tip.
6. Then,
7. rotate the coloured collar of the mixing tip 1/4 turn clockwise to the end of the cartridge. The dispenser isnow ready for use.
8. Squeeze the handle several times to extrude the material. After use, do not remove the MIXING TIP II as this will become the storage cap until next use. When replacing the MIXING TIP II, rotate the collar on the mixing tip 1/4 turn anti-clockwise to align the V-shaped notch on the cartridge. Tilt the mixing tip downward and peel itaway from the cartridge.
9. Remove and replace the old mixing tip immediately prior to next use. Before attaching a new tip, gently extrude a small amount of material to ensure that base and catalyst are flowing evenly from both openings. If the materials fail to extrude, remove any hardened materials on the end of thecartridge.
10. To replace the cartridge, lift the release lever and retract the plunger fully. Remove the empty cartridge by lifting the cartridge holder and load a new one into the dispenser.

IDENTIFICATION OF PARTS

- a. CartridgeHolder
- b. PistonPlunger
- c. Coloured Collar for MIXING TIP II
- d. ReleaseLever
- e. V-shaped Notch on the flange of the cartridge
- f. V-shaped Notch between the cartridgebarrels
- g. V-shaped Notch on the rim of MIXING TIP II

NOTES

1. Use only after checking that the cartridge and the mixing tip are loaded securely into the dispenser.
2. Sudden extrusion could cause cracking of the cartridge. To avoid this, gently squeeze the handle to extrude the material.
3. Clean the dispenser with an ethanol solution (or disinfecting) or dental wet tissue. The dispenser may be sterilised in an autoclave. However, with some autoclaves, the temperature of the sterilizing chamber may reach 132 °C or higher and there is a possibility that the dispenser material (synthetic resin) may become distorted at these temperatures. For details, contact the manufacturer of yourautoclave.
4. After autoclaving, do not use the dispenser until it has cooled down completely.

PACKAGES

1. CARTRIDGE DISPENSER II 1 piece

CE

GC

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.
Researchpark Hasserode-Leuven 1240, Interleuvenlaan33,
B-3001 Leuven, Belgium TEL : +32 16 74 10 00

PRINTED IN BELGIUM

2. MIXING TIP II SS (yellow), 60 pieces
3. MIXING TIP II S (pink), 60 pieces
4. MIXING TIP II LL (purple), 60 pieces
5. MIXING TIP II L (green / blue), 60pieces

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: https://www.gc.dental/europe. They can also be obtained from your supplier.

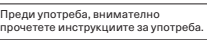
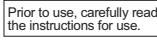
For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (https://ec.europa.eu/tools/eudamed) or contact us at Regulatory.gce@gc.dental.

Undesired Effects-Reporting:

If you become aware of any kind of undesired effect, reaction or similar events experienced by use of our product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to the internal vigilancesystem: vigilance@gc.dental

In this way you will contribute to improve the safety of this product.

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