

# TRANSPARENCY in your work



## EXACLEAR

Crystal Clear Vinyl Polysiloxane Material

### Indications

- Composite Injection Moulding Technique
- Temporary Crown and Bridge Work
- The Layering Technique
- Bite Registration
- Transfer of brackets or Fibers from the Model to the Mouth
- Fixing X-ray pellets used in Implant Planning

### Crystal Clear View



Dr. F. Klumpp, Germany

EXACLEAR: Visual control of composite infection.



Transparency of EXACLEAR (far right) in comparison to competitors.

### Optimal consistency for a successful Procedure

Flow without pressure



Optimal consistency for an **easy placement** in the tray

Flow under pressure



Fluid during insertion for a **perfect adaptation** to the preparation and **precise reproduction** of details

Easy to drill



**Rigid and strong** after setting and easy to cut and drill injection holes

Does not tear



**Flexible** enough for removal **without breakage or deformation**

Offers total Visual CONTROL



Facilitates **CONSISTENT** Result

**CREATING** outstanding aesthetic results

### Because Time also matters



only **2 minutes** intra-orally



Only **7 minutes** extra-orally

# Discover the Injection Moulding Technique

## Chairside creation of Direct Veneers

EXACLEAR used together with an injectable composite such as G-aenial Universal Injectable enables clinicians to create restorations as aesthetic as indirect veneers, but quicker, more affordable and using **less invasive preparations**.



1. Initial situation



2. Fill the impression tray with EXACLEAR and take an impression of the wax-up on the model



3. After setting, remove the clear silicone from the tray



4. Drill a hole through the silicone key to create space for the composite tip



5. Isolate the neighbouring teeth, etch the enamel and apply a bonding agent



6. Place the silicone key in the mouth and inject G-aenial Universal Injectable through the hole in the key



7. Light-cure through the silicone key and then remove it



8. Remove excesses of composite with a blade



9. Final result after a few polishing steps

## Your partners in complex clinical situations

### Injection Moulding Technique

Easy control of injection through EXACLEAR



#### With G-aenial Universal Injectable

Injectable composite with high strength and perfect thixotropy

### Layering Technique

Easy light-curing palatally through EXACLEAR



#### With G-aenial Anterior and SOLARE Sculpt

Highly aesthetic composite using a simplified shade selection

### Placement of Orthodontic Fibre Retainers

Perfect light-curing of bonding & fibres through EXACLEAR



#### With everStick ORTHO

Glass fibre reinforcement for aesthetic orthodontic retainers

### Temporary Crown and Bridge work

Perfect light-curing of the C&B material through EXACLEAR



#### With Tempsmart

Dual-cure temporary crown and bridge material

## Packaging

### Contains:

- 2 Cartridge Pack
- 2 x 48 ml Cartridge,
- 6 x Mixing Tips Size L

