Master all your challenges

*initial*TM
Lustre Pastes ONE

- EN Technical Manual
- DE Technische Arbeitsanleitung
- FR Manuel Technique
- IT Manuale tecnico

EN	DE	FR	IT	
Table of Contents	Inhaltsverzeichnis	Table des matières	Indice	Page
Intended Use	Anwendungsbereich	Utilisation	Campo di impiego	4-7
Description of the Lustre Pastes ONE	Beschreibung der Lustre Pastes ONE	Description des Lustre Pastes ONE	Descrizione delle Lustre Pastes ONE	8-11
The liquids of the Lustre Pastes ONE Set	Die Flüssigkeiten des Lustre Pastes ONE Set	Les liquides du coffret Lustre Pastes ONE	I liquidi del set di Lustre Pastes ONE	12-13
Lustre Pastes ONE Application	Lustre Pastes ONE Anwendung	Application Lustre Pastes ONE	Applicazione delle Lustre Pastes ONE	
Basic Colour Combination Chart	Kombinationstabelle Basis-Farben	Tableau de combinaisons des teintes Basic	Schema generale delle combinazioni cromatiche	14
Basic Step by Step Instruction	Basis-Farben Step by Step Anleitung	Instruction étape par étape	Istruzioni di base passaggio per passaggio	15
Advanced Colour Combination Chart	Individuelle Farb-Kombinationstabelle	Tableau de combinaison des teintes chromatiques	Schema avanzato delle combinazioni cromatiche	16
Advanced Step by Step Instruction	Erweiterte Step by Step Anleitung	Instruction étape par étape	Istruzioni avanzate passaggio per passaggio	17
Finishing	Finishing	Finition	Rifinitura	18
Firing Chart	Brenntabelle	Tableau des températures	Grafico delle cotture	19
Tips and Tricks	Tipps und Tricks	Trucs et astuces	Consigli e accorgimenti	20
Facilities of the Lustre Pastes ONE	Möglichkeiten mit Lustre Pastes ONE	Possibilités offertes avec les Lustre Pastes ONE	Esempi di applicazioni delle Lustre Pastes ONE	21
Physical Properties & Shelf Life	Physikalische Eigenschaften und Haltbarkeit	Propriétés physiques & Péréemption	Proprietà fisiche e durata utile del prodotto	22
FAQ	FAQ	FAQ	Domande più frequenti	23-25

GC Initial IQ Lustre Pastes ONE V-Shades

EN Intended Use

EN 3D paintable ceramics to bring depth, authenticity and gloss

GC Initial IQ Lustre Pastes ONE (LP ONE) is a feldspar based ceramic material, available in a ready to use consistency.

The Lustre Pastes ONE are suitable for a wide CTE-range - both 'low' and 'high' CTE ceramics - with an optimal match to the complete GC Initial ceramic range and nearly all types of other ceramics (see 'Indications' and page 16) Composed of fine ceramic particles, the 3-dimensional paint-on pastes bring colour deepness and lifelike translucency to your restorations. They can be applied in a thicker layer guaranteeing unsurpassed vitality and a very natural glaze.

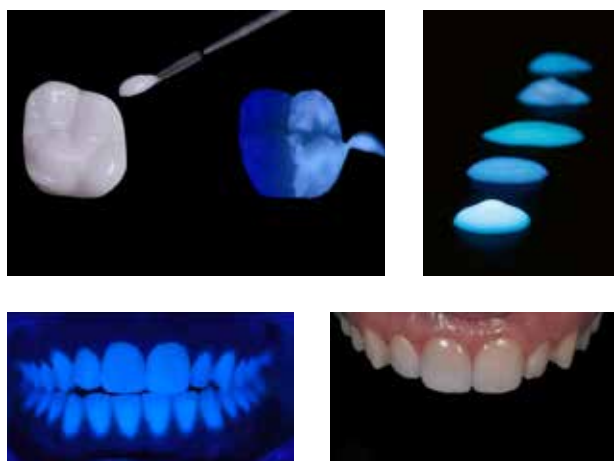
Improved fluorescence for all your monolithic restorations

Looking for a way to add extra vitality to your CAD/CAM produced zirconia monolithics?

Lustre Pastes ONE are the perfect solution for all your restorations!

The Lustre Pastes ONE, with improved fluorescence, are especially developed to boost the fluorescence of your monolithic restorations.

All Lustre Pastes ONE are fluorescent, throughout different levels. Boost your fluorescence level with Lustre Neutral FLUO (L-NFL).

Picture courtesy MDT Leonardo Cavallo, Italy

Indications

The Lustre Pastes ONE represent the newest alternative for conventional staining and glazing.

Lustre Pastes ONE offer you as well a wide variety of possibilities to adapt colour, brightness, grey value and surface texture corrections to all your ceramic crown & bridge restorations, both monolithic as layered.

Due to its unique properties, the Lustre Pastes ONE are the most versatile ceramic material to individualize your milled and pressed monolithic restorations or to internally characterize the substructure for the micro layering technique in combination with the Initial SQIN powders. Likewise, traditional veneering ceramics can be painted and glazed in no time.

The Lustre Pastes ONE are suited for the internal & external characterization of:

- Zirconia frameworks (e.g. Zirconia Disk, ...)
- Lithium DiSilicate frameworks (e.g. LiSi Press, ...)
- Layering ceramics with CTE between 6,9 and 13,3 (e.g. Initial MC, Initial Zr-FS, ...)

Keep in mind:

What you paint on is what you get after firing.

Features & benefits

- Just one dedicated kit for all different indications for the complete GC Initial ceramic range.
- The Lustre Pastes ONE are based on a mix of refined glass ceramic particles, ensuring lifelike depth light dynamics for a natural 3D effect.
- Improved and unique paste medium with a fine thixotropic consistency enabling easier and more precise application properties compared to conventional stains.
- Ready to use. Adaptable to your preferred consistency.
- Optimal fluorescence ensure the look of natural teeth under all light circumstances.
- GC Initial Spectrum Stains (powder stains) can be used to increase individualization possibilities.
- Perfect combination with Initial SQIN for the full ceramic micro layering technique.

Contents:

1x Neutral FLUO, 1x Neutral, 4x Body Shades, 10 x Enamel Effect shades, 1x Effect Opal, 1x Effect Value, Diluting Liquid, Refreshing Liquid, 1 Mixing dish, 2 Brushes, Printed shade chart, IFU.

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EN Note: Mix the material well before use.

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EN Lustre Paste "Diluting Liquid":

The 'Diluting Liquid' is a special liquid to change the flowability of the Lustre Pastes. However, it is recommended to obtain a gel/paste like consistency of the Lustre Pastes for best results.

Attention:

- If the Lustre Pastes are too much diluted, the result is a reduction in the gloss of the paste after the firing; hence, the desired result may not be achieved in one glaze firing.
- When needed, dilute the desired amounts of the pastes in a separate plate with Diluting Liquid, **never dilute directly in the jars.**

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EN **Lustre Paste “Refresh Liquid”:**

The ‘Refreshing Liquid’ is used to preserve the thixotropic property of the pastes. This liquid is used to recover dried out pastes to an optimum, gel like consistency without changing the physical properties.

The Refresh Liquid can be used both in the jars and mixed on a separate plate.

Important: Please shake the Refresh Liquid well before use!

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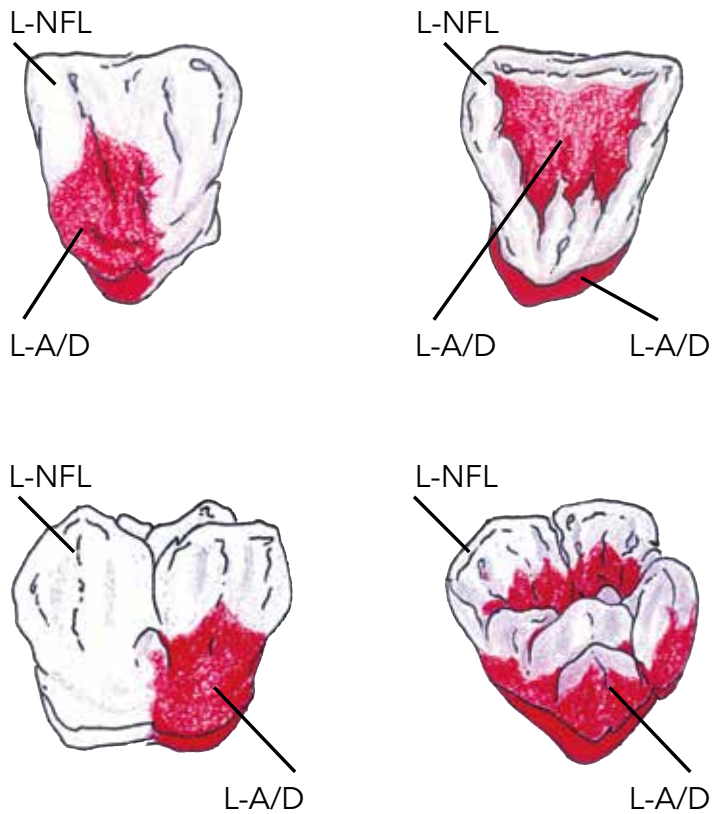
EN Important Information: Please always stir up material well in the jars before use. Before applying the Lustre Coating, the ceramic restoration is shaped and contoured using adjusted diamond burs. After prepolishing with a ceramic polisher the finished crowns & bridges are sand-blasted with 50 microns Alu-Oxid at 1,5 bar pressure.

The Lustre Paste Neutral Fluo (L-NFL) is coated on the entire surface of the sandblasted restoration. This coating is clearly thicker than what we know from a normal glaze firing.

Note: These Lustre Pastes may not come in contact with water.

The requested V-Classic Shade is created by a more or less intensive application of the Lustre Pastes "Body A - D". This process, which only takes 2 minutes, is everything which is necessary to create V-Classic shades.

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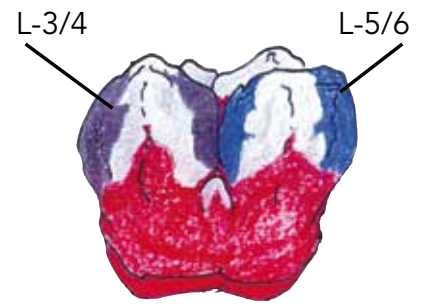
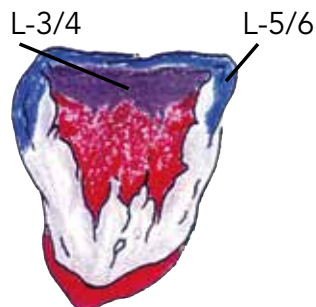
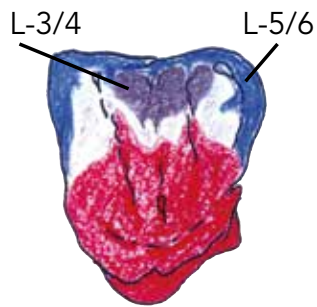
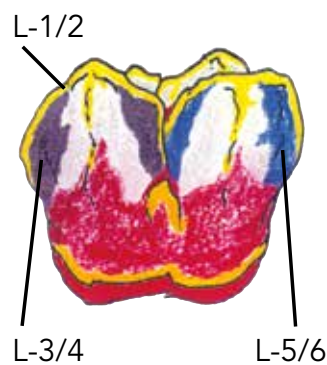
- EN To build up a high aesthetic solution for the patient use our advanced step by step instruction:
In order to achieve the higher grey value of the incisal area with darker teeth (A3.5; A4; C3; C4), 2 grey Effect Shades can be used: L-3 (Light Grey) and L-4 (Dark Grey). For eventual further individualisation, you can use the whitish Lustre Pastes ONE: L-1 (Vanilla) and L-2 (White) and L-V (Value) for brightening the restorations. 2 bluish Effect Lustre Pastes ONE: L-5 (Light Blue) and L-6 (Dark Blue) for a bluish opalescent appearance in the incisal area. Alternatively, Lustre Opal (L-OP) can be used in the incisal area for an optimal opalescent result. Greenish Effect Lustre Paste NF: L-8 (Olive) for a greenish effect. Can be mixed with all Lustre Pastes ONE. In order to achieve an even higher „Transparency“ the 2 greyish Effect Lustre Pastes ONE L-3/L-4 can be used as well. All GC Initial Spectrum Stains can be used to achieve even more possibilities of individualization.

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EN The requested surface texture / smoothness of the Lustre Paste layer can be obtained by soft vibrating or condensing the restoration.

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EN No matter if you choose the basic way or the advanced way for high aesthetic results, the steps to finalize your work are still the same.

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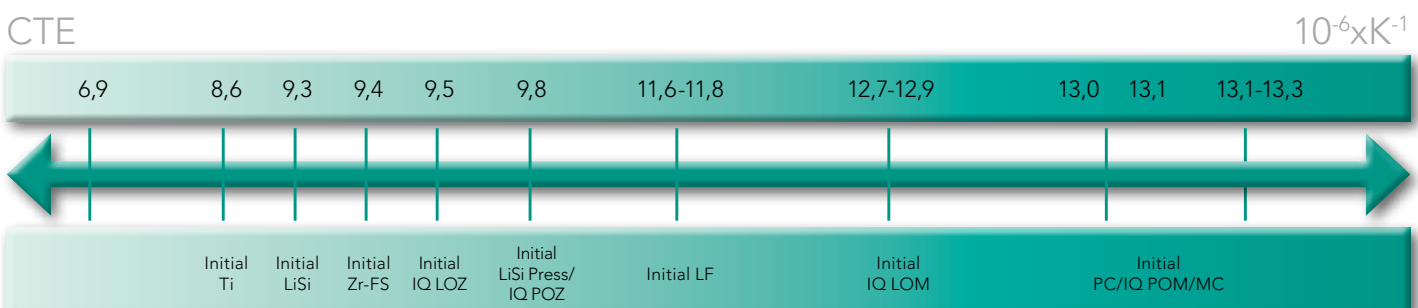
EN Tips and Tricks

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EN Additional changing's in colour, transparency and brightness of crown and bridge work can be done smoothly with the Lustre Pastes. The unique consistency ensures homogenous modifications and appearance.

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EN Make your corrections with the effect colours of the Lustre Paste Set.

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Basis
Base

LP Neutral Fluor

EN Use the Lustre Pastes to inhibit the depth effects and the transparency of your crown and bridge work build in layering technique. Sandblast your restoration with 50 micron Aluminumoxide, damp it and then paint it on with Lustre Paste L-NFL.

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EN The requested surface texture/smoothness of the Lustre Pastes layer can be obtained by soft vibrating or condensing the restoration.

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Classification: Dental Ceramics Type I / Class 1

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DE FAQ

FR FAQ

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EN **Basic Rule: What you paint (on) is what you get (after firing)**

- Q1. How can I dilute (adapt the consistency of) the Lustre Pastes ONE?**
Use the Lustre Pastes ONE Diluting Liquid to adapt the consistency. Do NOT use water to dilute. LP may not come in contact with water (see page 12).
- Q2. What can I do when the consistency of the Lustre Pastes ONE is too dry?**
Use the Refresh Liquid to recover the Lustre Pastes ONE in case of dryness (see page 12).
- Q3. How do we change the texture of the Lustre Pastes ONE?**
The requested surface texture/smoothness of the Lustre Pastes ONE layer can be changed by soft vibration or condensing the applied pastes.
- Q4. I do not have a shiny appearance after firing the Lustre Pastes ONE, what is the cause?**
Check the firing temperatures, increase the final temperature. When the Lustre Pastes ONE are heavily diluted or applied in a too thin layer the glossy effect is tempered, so apply a more thick layer/dilute less the pastes.
- Q5. Can I apply more than one layer of Lustre Pastes ONE?**
Yes, you can do further firings, keeping the same firing temperatures.
- Q6. Can I mix the Lustre Pastes ONE with other Initial powders?**
You can use the GC Initial Spectrum Stains powders with moderation to intensify the Lustre Pastes ONE. Mix the required amount of stain powder with the LP ONE Diluting Liquid and mix it with the required ONE paste or directly mix the Neutral FLUO L-NFL.
- Q7. I have finished my veneered crown (SQIN) and I want to use the Lustre Pastes ONE without losing the surface texture of the prepared crown.**
You can easily dilute the LP to have the optimal consistency in order not to lose the surface texture and still have a nicely coloured and shiny crown, just apply in a thin layer.
- Q8. Are the Lustre Pastes ONE sufficient abrasion resistant?**
Yes, the LP ONE are a special mixture of different types of low fusing ceramic particles, stains & glaze mixture offering colour deepness and lifelike translucency.
- Q9. I have to change the colour, brightness or the grey value of a crown made with the layering technique. What can I do?**
Use the Lustre Pastes ONE as shown on page 20 to modify your crown and bridge work.

Q10. After firing the colour of my work is spotted?

The Body Colour needs to be massaged into the Lustre Paste Neutral Fluo (L-NFL) on your crown or should be mixed equally on an external mixing board in advance. To disperse the colour pigments it is necessary to condensate the Lustre Pastes ONE before the glaze firing by soft vibration.

Q11. The colour doesn't fit with the V-Classic Shade guide?

Different material thicknesses cause different colour effects. Therefore, it may be necessary in individual cases, to mix an A-Body Colour with a B-Body or even with L8 (Olive). A visual check is always possible.

Q12. The glaze firing is blearily, spotted or cracked?

The Lustre Pastes ONE came partly or completely in contact with water.

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GC NORDIC AB - Finnish Branch
GC NORDIC AB - GC Nordic Danish Branch
GC NORDIC AB
GC AUSTRIA GmbH - Swiss Office
GC AUSTRALASIA DENTAL PTY LTD
GC ASIA DENTAL PTE. LTD.