Erkodur /klar/240/freeze/0M1/-A1/-A2/-A3/Usig

23°F
-5°C
45°C
113°F

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The general purpose of Erkodur is the manufacture of appliances for dental and orthodontic treatment.

Application (material)	Intended use	Indication	Patient groups
Aligner (Erkodur)	Positioning of teeth	Misaligned teeth	Adults, teenagers
Bite splint (all)	CMD treatment	CMD	Adults, teenagers, children
Friction sleeve (Usig)	Support for attachment and telescope technology	Dentures with attachment and telescope technology	Adults
Planning template (Erkodur)	Implant planning	missing teeth	Adults, teenagers
Temporary appliances (Erkodur, Erkodur tooth-colored)	Maintain chewing function with protective function	Interim supply	Adults, teenagers
Protective splints (Erkodur)	Protection after local surgery	Local surgery (e.g. implantation)	Adults, teenagers
Silensor-sl - snoring protection (Erkodur, Erkodur freeze)	Reduction or elimination of ronchopathy and OSA	Ronchopathy and OSA	Adults, teenagers
Stabilizing splint (Erkodur, Erkodur tooth-coloured)	Stabilization of tooth positions	Misaligned teeth	Adults, teenagers
Bandage plate (Erkodur)	Cover, hold or press on oral wound care	Wound care for oral wounds	Adults, teenagers, children

Contraindications: If an allergic reaction to plastics is already known, a specific allergy to Erkodur (PETG) should be ruled out.

Caution: Sensitive to moisture, close the aluminum bag tightly again immediately. Observe storage symbols.

User: Dental professionals

Intended use: Erkodur is recommended as a medical device exclusively for the above-mentioned purposes.

Note: The use is subject to responsibility and is based on the recommendation/prescription of the dentist. Allergic reactions are unlikely, but cannot be ruled out. Incorrect processing can lead to cracks, fractures and sharp edges. There is a risk of aspiration in the case of small fractures.

Combination products: Erkodur can be processed in all devices of the Erkoform and Erkopress series from Erkodent.

Cleaning: Dental splints can be cleaned with Oxydens cleaning tablets. Also with curd soap, water and a toothbrush. Toothpaste is not suitable. Do not heat above 50 °C. Please rinse the splint with clear water before inserting it. As the user, please inform the patient about the cleaning options.

Further information: Thermofforming technique brochure and videos, www.erkodent.com

Serious incidents in connection with Erkodur must be reported. The user must inform the competent authority and the manufacturer.

Performance characteristics (Circa values): Density 1.27 g/cm³, modulus of elasticity 2276 MPa, elongation at break 40 %, water absorption 0.13 %

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