GC KommonBase™

LIGHT-CURED FLOWABLE RESIN FOR ORTHODONTICS

For use only by a dental professional in the indications for use.


INDICATIONS FOR USE
Fabricating resin base for bracket

CONTRAINDICATIONS
Avoid use of this product in patients with known allergies to methacrylate monomer or methacrylate polymer.


COMPOSITION
HV
Dimethacrylate, barium glass, silicon dioxide, phosphoric acid ester monomer, initiator, pigment, stabilizer
LV
Dimethacrylate, barium glass, phosphoric acid ester monomer, silicon dioxide, initiator, pigment, stabilizer

DIRECTIONS FOR USE
A. MODEL AND BRACKET PREPARATION
1. Soap the setup stone model to facilitate the removal of GC KommonBase from the stone model.
2. Make sure that the bracket is pretreated and handled according to the manufacturer's instructions. Place the brackets to the reference wire.

B. SYRINGE PREPARATION
1. Hold the syringe upright and remove the cap by turning counterclockwise.
2. Promptly and securely attach the dispensing tip to the syringe by turning clockwise.
3. Place the light protective cover until use.
Take care not to attach the dispensing tip too tightly. This may damage its screw. In order to ensure a tight connection make sure that the threads are free of remnant material.
Note: Do not use too excessively thick the base.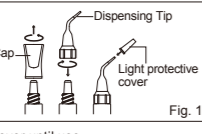

C. FABRICATION OF RESIN BASE
1. Remove cover from the dispensing tip.
Note: Prior to syringe material, make sure that dispensing tip is securely attached to the syringe. Any possible air from the dispensing tip is removed by gently pushing forward the syringe plunger with the tip pointing upwards until material reaches the opening of the tip (Fig. 2).
2. Slowly push the plunger to syringe material. If plugged, remove the dispensing tip and extrude a small amount of material directly from the syringe.
3. Apply the GC KommonBase paste to the adherent surface of bracket and spread with a brush to make the resin base.

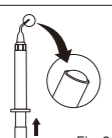4. Position the brackets with the paste onto the setup stone model.
5. Light cure the paste with the brackets located on the stone model. The irradiation time is 10 seconds for each bracket with an LED light curing device (>500 mW/cm², 460-480 nm).
Note: It is very important that the curing light be tested periodically with a light meter to ensure adequate light output.
6. Add the GC KommonBase paste with an appropriate instrument and extend the resin base on the stone model.
Be careful not to excessively thick the base.
7. Light cure the extended resin base for 10 seconds each with two directions with an LED light curing device (>500 mW/cm², 460-480 nm).
8. Detach the brackets with the resin base from the stone model using tweezers. Remove each bracket from the reference wire.
9. Adjust the resin base on the original stone model.
When the resin base interferes with each other or opposing or adjacent teeth, grind the interfering area with a diamond bur. 10. Blend the adherent surface of the resin base with anhydrous alcohol. The resin base is ready to be bonded.
11. Clean the resin base to tooth surface using dental cement such as GC Ortho Connect or resin-modified glass ionomer cement. For the treatment of tooth surface and bonding procedure, follow the manufacturer's instructions for use of the selected cement.

FLOW & SHADE
HV: Clear
LV: Pink

STORAGE
Recommended for optimal performance, store at room temperature (4-25°C / 39-277.0°F) away from direct sunlight.

PACKAGES
2 x 2.7 g (2.0 mL) syringes x GC KommonBase,
4 x dispensing tip for GC KommonBase,
1 light protective cover

CAUTION
1. In case of contact with oral tissue or skin, remove immediately with cotton or a sponge soaked in alcohol. Flush with water.
2. In case of contact with eyes, flush immediately with water and seek medical attention.
3. Take care to avoid ingestion of the material.
4. Do not mix with other similar products.
5. Wear plastic or rubber gloves during operation to avoid direct contact with air inhibited resin layers in order to prevent possible sensitivity.
6. Wear protective eye glasses during light curing.
7. When polishing the polymerized material, use a dust collector and wear a dust mask to avoid inhalation of cutting dust.
8. Avoid getting material on clothing.
9. Do not wipe the dispensing tip with excessive alcohol as this may cause contamination of the paste.
10. Dispose of all wastes according to local regulations.
11. In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
12. Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at: www.gc.dental/ortho
They can also be obtained from your supplier.

CLEANING AND DISINFECTING
MULTI-USE DELIVERY SYSTEMS: to avoid cross-contamination between patients this device requires mid-level disinfection immediately after use inspect device and label for deterioration. Discard device if damaged.
DO NOT IMMERSER. Thoroughly clean device to prevent drying and accumulation of contaminants. Disinfect with a mid-level registered healthcare-grade infection control product according to regional / national guidelines.

For the Summary of Safety and Clinical Performance (SSCP) please see EUDAMED database (<https://ec.europa.eu/tools/eudamed>) or contact us at Regulatory.gce@gc.dental

Undesired effects- Reporting:
If you become aware of any kind of undesired effect, reaction or similar events experienced by use of this product, including those not listed in this instruction for use, please report them directly through the relevant vigilance system, by selecting the proper authority of your country accessible through the following link: https://ec.europa.eu/growth/sectors/medical-devices/contacts_en as well as to our internal vigilance system: vigilance@gc.dental. In this way you will contribute to improve the safety of this product.

UK Responsible Person
GC UNITED KINGDOM Ltd.
Coopers Court Newport Pagnell, Buckinghamshire, MK16 8JS
United Kingdom

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